
TEMPERATURE PACK

Instructions for use | Gebrauchsanweisung | Mode d'emploi | Istruzioni per l'uso |
Instrucciones de uso | Indicações de utilização | Gebruiksaanwijzing | Bruksanvisning |
Käyttöohje | Bruksanvisning | Brugsanvisning | Инструкция по эксплуатации |
Οδηγίες χρήσης | Kullanım Kılavuzu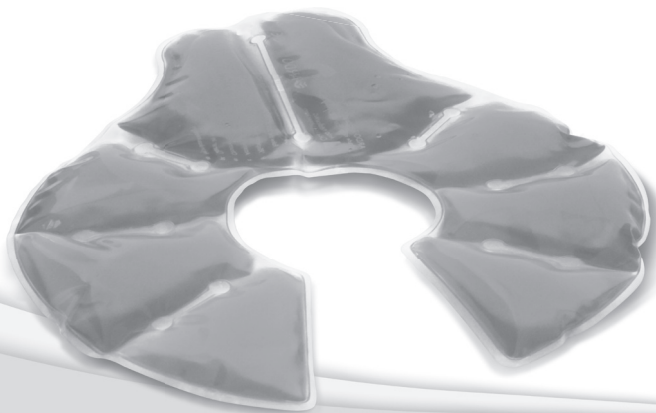

	Languages		Page
en	Instructions for use	Temperature Pack	4
de	Gebrauchsanweisung	Temperature Pack	5
fr	Mode d'emploi	Temperature Pack	6
it	Istruzioni per l'uso	Temperature Pack	7
es	Instrucciones de uso	Temperature Pack	8
pt	Indicações de utilização	Temperature Pack	9
nl	Gebruiksaanwijzing	Temperature Pack	10
sv	Bruksanvisning	Temperature Pack	11
fi	Käyttöohje	Temperature Pack	12
no	Bruksanvisning	Temperature Pack	13
da	Brugsanvisning	Temperature Pack	14
ru	Инструкция по эксплуатации	Temperature Pack	15
el	Οδηγίες χρήσης	Temperature Pack	16
tr	Kullanım Kılavuzu	Temperature Pack	17

INSTRUCTIONS FOR USE

TEMPERATURE PACK

ANATOMICAL BREAST COMPRESS

Package contains

Ardo Temperature Pack, soft cover, instructions for use

Intended use

Breast compress for cold and warm application to breasts during breastfeeding. As a warm compress it helps to trigger the let-down reflex and as a cold compress it can relieve breast inflammation, the build-up of milk or tension in the breast. Its anatomical shape means it can be placed under the armpit and used as a Temperature Pack for the lateral breast glands. The round opening prevents contact with sensitive nipples.

Hold the Temperature Pack comfortably in place on the breast, preferably while lying down. Use the fastener of the cover to adapt it to your breast size. To warm/cool the lateral breast glands, hold the flap lightly under your arm.

Warm compress application: put on approximately 5 minutes before breastfeeding as described. The warmth promotes a feeling of relaxation and helps to trigger the let-down reflex.

Heating up in warm water: boil the water without the Temperature Pack. Place the Temperature Pack without cover into water (not more than 80 °C) and for no more than 10 minutes. Caution! The Temperature Pack can burst if overheated. The colour of the Temperature Pack foil may change when heated. Dry the Temperature Pack with a towel.

Heating up in microwave: place the Temperature Pack without cover flat on a plate and spread the gel out evenly. Caution! The Temperature Pack can burst if overheated. Cover the Temperature Pack with a microwave lid to protect the microwave from possible contamination. Heat for no more than 30 seconds at 800 watt. The colour of the Temperature Pack foil may change when heated. Take the Temperature Pack out of the microwave and carefully test the temperature. If necessary, heat the Temperature Pack further, in small steps, until the desired temperature is reached (for a total of no more than a maximum of 1.5 minutes at 800 watt).

Cold compress application: put on for approximately 15–20 minutes after breastfeeding as described. The coolness provides soothing relief for breast inflammation, engorgement or tension in the breast.

Cooling in a freezer compartment: spread the gel out evenly. Place the unfolded Temperature Pack without cover flat in the freezer compartment (not colder than -18 °C) for no more than 1 hour. Do not fold when frozen!

Cleaning instructions

Wash your hands thoroughly before each use. Before first use and after each use, wash the Temperature Pack in soapy water and then rinse it thoroughly. In the hospital: wipe with disinfectant. Do not sterilize! Soft cover: non-washable and for use by one person only.

Hints and contraindication

Never use the Temperature Pack without the soft cover to avoid burning the skin. The soft cover is non-washable and only suitable for use by one person. Caution! Do not use together with skin care oil to avoid potential damage to the foil! Do not use a faulty Temperature Pack! Do not store in a freezer compartment or heating unit! Keep small parts and plastic bags out of the reach of children! The Temperature Pack and its contents are non-toxic. Stop using the Temperature Pack and seek specialist advice if it is unpleasant to use or causes pain.

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Ardo medical AG